


this pen.

As with all health care devices, it is recommended that you get training from your health care professional prior to using

The HumaPen® Savvio™ comes with a comprehensive user manual, which should be read fully before use.

The HumaPen® Savvio™ comes with a comprehensive user

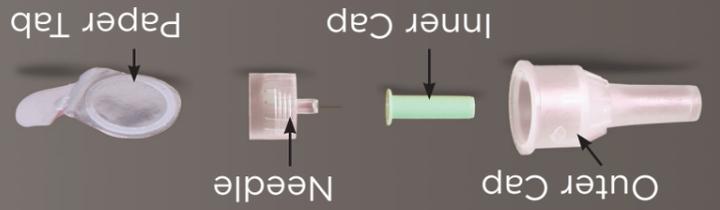


Needle (available separately)



in the Step by Step guidance

Cloudy insulin must be mixed before use. This is explained

Insulin Cartridge

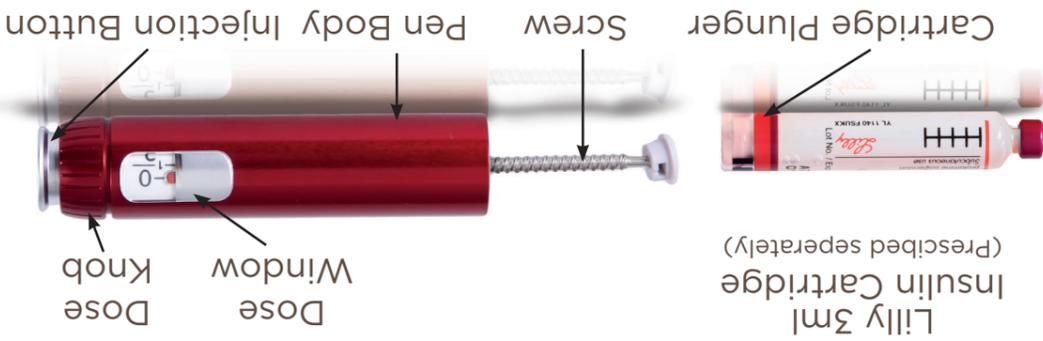

HumaPen® savvio™

HumaPen® savvio™

HumaPen® savvio™



A simple guide to using your HumaPen® Savvio™

If you experience side effects talk to your doctor or other healthcare professional. This includes any possible side effects not listed in the package leaflet. To report a side effect or product complaint with a Lilly product please call Lilly on +44 (0) 1256 315000 (UK) or +353 (0)1 664 0446 (ROI).

Additionally reporting forms and further information can be found at www.mhra.gov.uk/yellowcard or search for MHRA Yellow Card in the Google Play or Apple App Store (UK) or www.hpra.ie (ROI). By reporting side effects you can help provide more information on the safety of medicines.

This guide is for patients who have been prescribed a HumaPen® Savvio™.

PP-LD-GB-0282 July 2018

How to use your Pen

1 Inserting the insulin cartridge




2 Mixing Cloudy Insulin

If your insulin is cloudy it is important to mix it




3 Attaching the needle

A new needle should be used before every injection

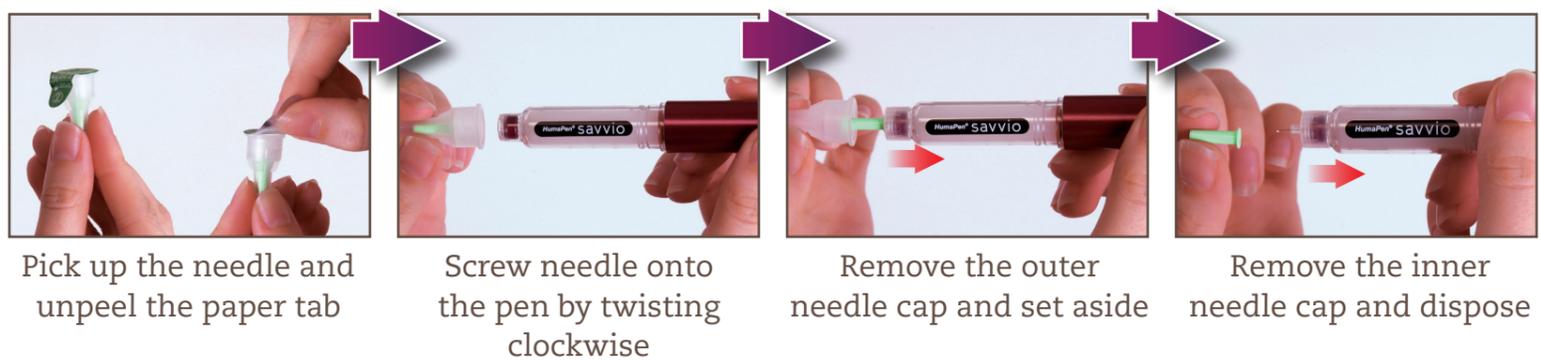

4 Priming the Pen

You should prime your pen before every injection

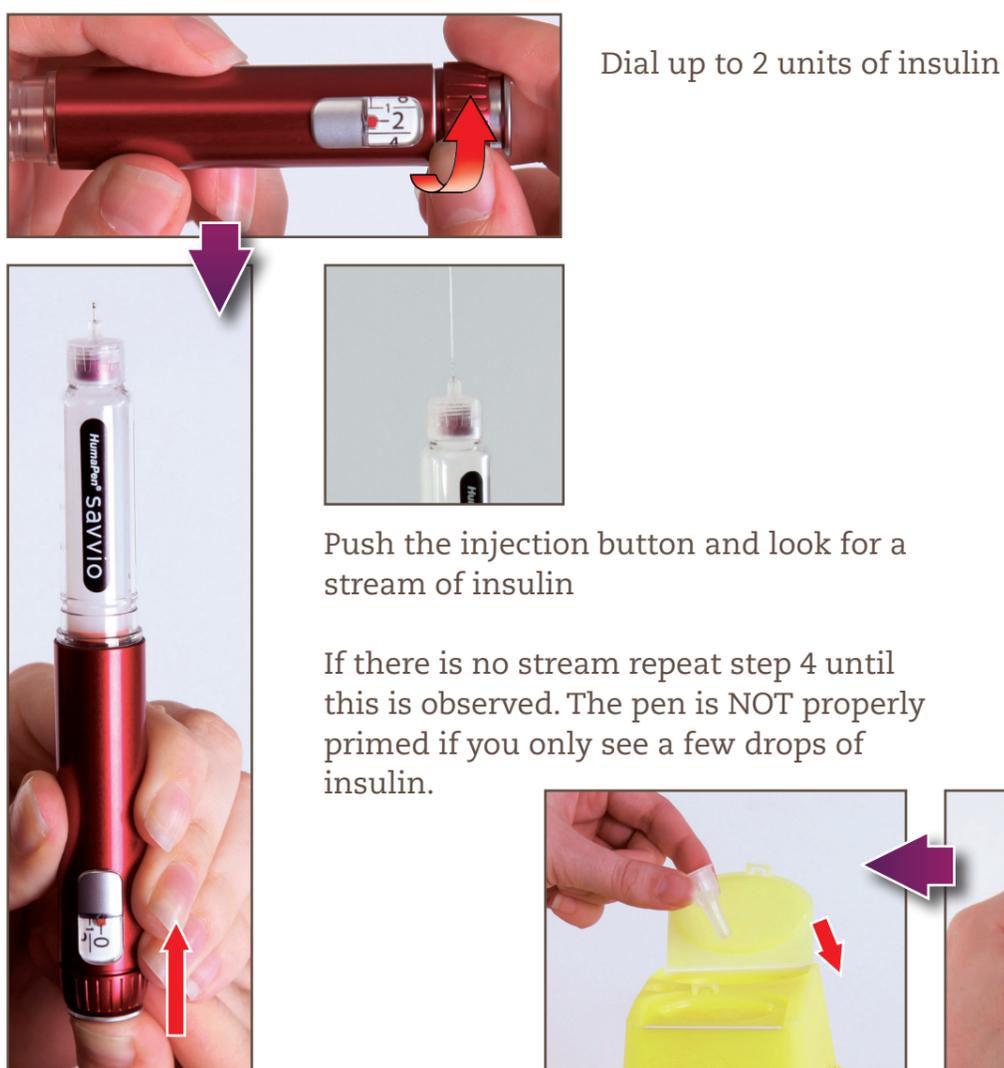

5 Injecting the dose

