



A simple guide to using your KwikPen®

Important Information:

- The KwikPen comes with a comprehensive user manual, which should be read fully before use.
- Spare KwikPens should be kept in the fridge until ready for use.
- Your 'in-use' KwikPen can be kept at room temperature for up to 28 days.
- Prime the pen before each injection to check insulin flow.
- Don't leave a needle on your KwikPen between injections.

If you experience side effects talk to your doctor or other healthcare professional. This includes any possible side effects not listed in the package leaflet. To report a side effect or product complaint with a Lilly product please call Lilly on **+44 (0) 1256 315000 (UK) or +353 (0)1 664 0446 (ROI)**.

Additionally reporting forms and further information can be found at **www.mhra.gov.uk/yellowcard** or search for MHRA Yellow Card in the Google Play or Apple App Store (UK) or **www.hpra.ie** (ROI). By reporting side effects you can help provide more information on the safety of medicines.

PP-LD-IE-0014 June 2018

How to use your pen

Step 1: Prepare pen for use

1. Wash your hands
2. Pull the Pen Cap off to remove it




3. If your insulin is cloudy, gently roll and turn the pen 10 times to mix it

Step 2: Attaching the needle

(a new needle should be used before every injection)

4. Pick up the needle and unpeel the paper tab
5. Screw needle onto the pen by twisting clockwise
6. Remove the outer needle cap and set aside
7. Remove the inner needle cap and dispose

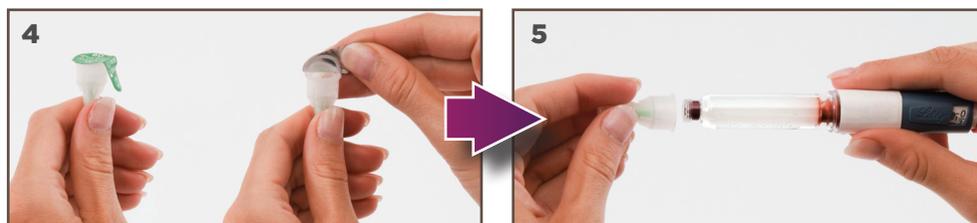

Step 3: Priming the pen

8. Look at the dose window and dial 2 units. If you dial too much, just dial back.
9. Push the injection button and look for a stream of insulin

If there is no stream repeat step 3 until this is observed

Step 4: Injecting the dose

10. Look at the dose window and dial your agreed dose of insulin. Odd numbers show as lines between even numbers. If you dial too many units, dial backwards to correct
11. Insert needle as demonstrated by your doctor/nurse. Place your thumb on the injection button. Push firmly until it stops. Hold for 5 seconds
12. Remove needle slowly and check dose window reads "0"
13. Replace the outer cap and whilst grasping the cartridge holder, unscrew the capped needle.
14. Dispose of as directed by your healthcare professional

